

AUTHORIZED VAC SEW

STEP 1- CUT TWO STRIPS 1 1/4" X WIDTH OF FABRIC

FOLD EACH LONG SIDE TO THE MIDDLE, THEN FOLD ONE MORE TIME LENGTHWISE

SEW USING A ZIG ZAG OR STRETCH ZIG ZAG

CUT EACH STRAP IN HALF. THEY WILL BE 21" LONG

STEP 2- CUT TWO PIECES OF THE PATTERN ON THE FOLD

STEP 3- SEW TWO STRAPS ON EACH SIDE (SEE DIAGRAM BELOW) SO THAT THEY PEAK OUT JUST A BIT FROM THE MASK, BUT THE STRIP EXTENDS INWARDS

STEP 4- ZIG ZAG A 4" PIECE OF PIPE CLEANER TO CENTER TOP SIDE OF THE LINING PIECE ABOUT 3/8" DOWN

STEP 5- PLACE BOTH MASKS RIGHT SIDES TOGETHER. USING A 1/4IN SEAM ALLOWANCE SEW AROUND ENTIRE MASK LEAVING A 3" OPENING AT THE BOTTOM TO TURN THE MASK


STEP 6- TURN THE MASK

STEP 7- PRESS THE MASK WHILE TURNING IN THE OPENING TO THE INSIDE, TOP STITCH AROUND THE ENTIRE MASK

STEP 8- SEW 3 TUCKS ON EACH SIDE AND TWO AT THE BOTTOM OF THE MASK, REFER TO PATTERN FOR PLACEMENT OF TUCKS

THANK YOU FOR SUPPORTING OUR SHORTAGE OF MASKS
PLEASE DROP OFF MASKS SO THEY CAN BE PROPERLY
SANITIZED AND WE WILL DISTRIBUTE THEM TO OUR LOCAL MEDICAL FACILITIES.

AUTHORIZED VAC AND SEW HAS BEEN SERVING OUR COMMUNITY SINCE 1975 WE APPRECIATE
ALL YOUR LOVE AND SUPPORT

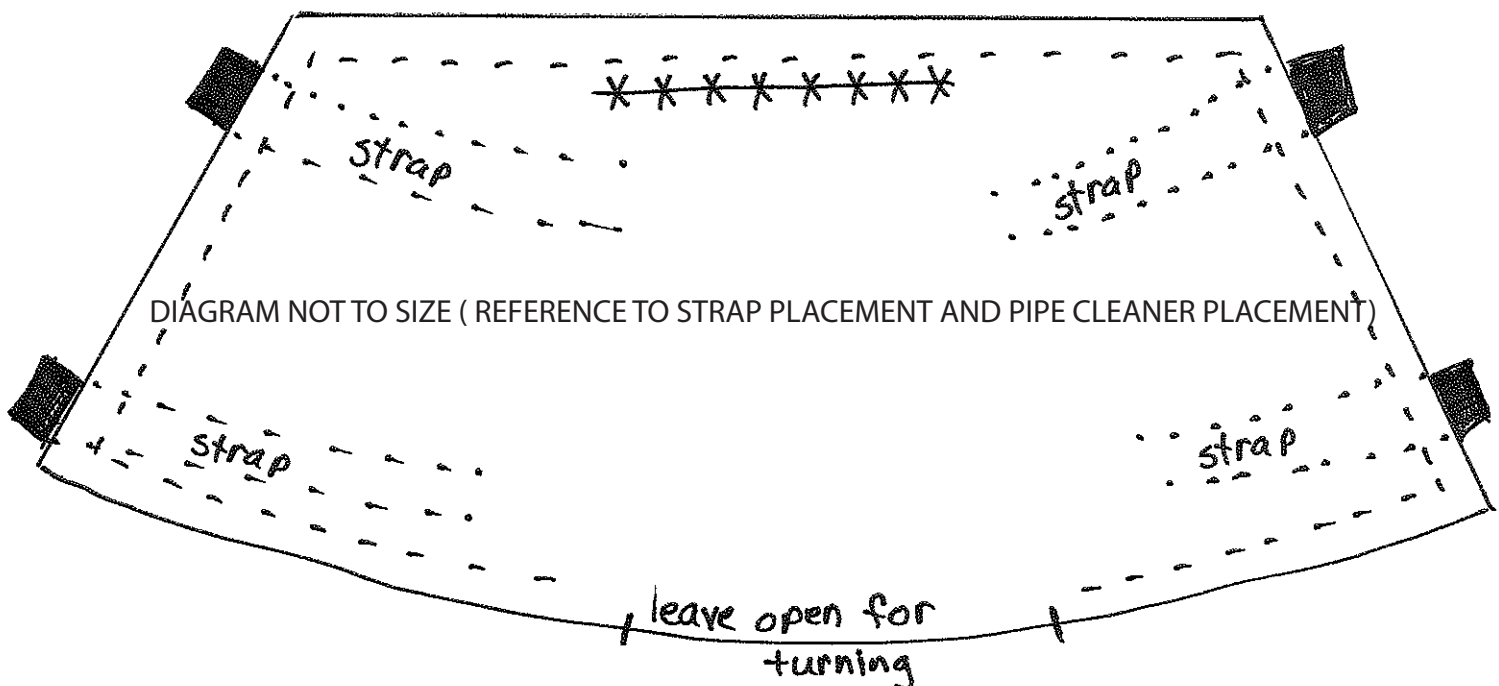

Top of Mask

